


Visual & Electronic Tags Handling & Tagging Instructions




1 Load female first
Secure tag below clip

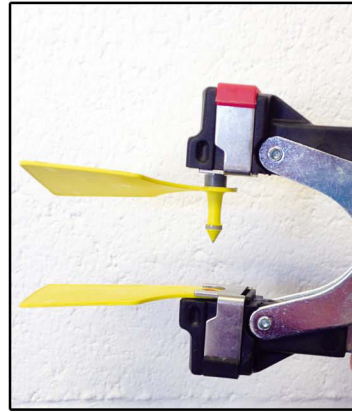

2 Load male second
Push onto pin firmly



3 Insert tag between two cartilage ribs
Close applicator fully
Audible click is heard!

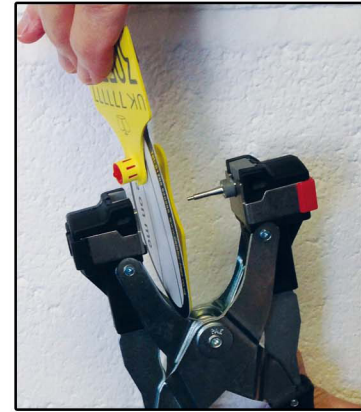


4 Open applicator fully to release tag

Important tagging information


Always ensure that the female part of the tag is inside the ear

✓ Reduces the chance of the tag snagging or catching

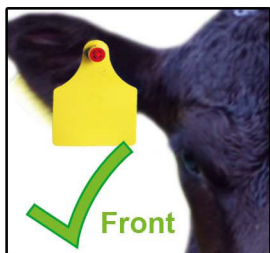


Insert the tag between the two cartilage ribs.


✓ Recommended position

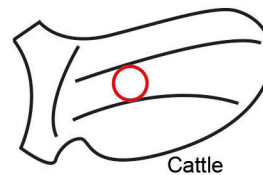

Cattle

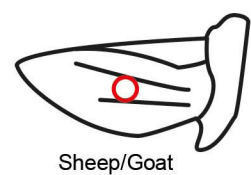

Sheep/Goat

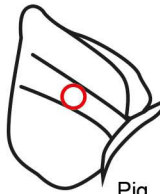

Pig

Apply tags under hygienic conditions

- ✓ Operator hands, the ear, the tag and the applicator should be clean
- ✓ Disinfect as necessary using an appropriate solution
- ✓ Fit in cool weather (where possible) to minimise infections
- ✓ Store unused tags in clean dry container

When tagging animal from the front use the hole in female jaw of applicator as a target

- ✓ Straight-line applicator action drives male & female precisely together
- ✓ Eliminates using male point to locate tag correctly in ear



Check the ear after about 10 days for signs of infection

- ✓ Consult your veterinary surgeon as necessary

Always use correct applicator

Use Combo Strips

- ✓ Speeds up tagging
- ✓ Avoids inserting non-matching tags

